Aplikacija Unitron Remote Plus

Korisničko uputstvo

Prvi koraci

Namena

Aplikacija Unitron Remote Plus je namenjena korisnicima slušnih aparata kojima omogućava da određene aspekte slušnih aparata kompanije Unitron podese putem Android ili Apple iOS uređaja¹:

Ako specijalista za slušna pomagala korisniku slušnih aparata ponudi **Insights** (Uvidi) funkcije i korisnik pristane na njihovo korišćenje, korisnik može da šalje podatke i povratne informacije o iskustvu korišćenja slušnog aparata, dok specijalista za slušna pomagala može daljinski da podešava slušni aparat.

Informacije o kompatibilnosti:

⚠ Za korišćenje aplikacije Unitron Remote Plus potrebni su bežični Bluetooth slušni aparati kompanije Unitron. Aplikacija Unitron Remote Plus može da se koristi na uređajima sa tehnologijom Bluetooth® Low-Energy (BT-LE) i kompatibilna je sa operativnim sistemom iOS verzije 12 ili novije. Aplikacija Unitron Remote Plus može da se koristi na Android uređajima certifikovanim za usluge Google Mobile Services (GMS) koji podržavaju Bluetooth 4.2 i Android OS 7 ili noviji.

- ① **Pojedini telefoni emituju zvučni signal pri dodirivanju ekrana ili tastature i taj signal potom može da se prenese na slušne aparate.**
Da biste izbegli to, pristupite podešavanjima telefona, izaberite audio-podešavanja i uverite se da su deaktivirani svi zvučni signali pri dodirivanju ekrana ili tastature.

¹ Kompatibilni telefoni: Aplikacija Unitron Remote Plus može da se koristi samo na telefonima sa Bluetooth® low energy tehnologijom.

Reč Bluetooth® i logotipi su registrovani zaštitni žigovi u vlasništvu kompanije Bluetooth SIG, Inc.

Apple, Apple logotip, iPhone i iOS su zaštitni žigovi kompanije Apple Inc. registrovani u SAD i drugim zemljama. App Store je servisna marka kompanije Apple Inc.

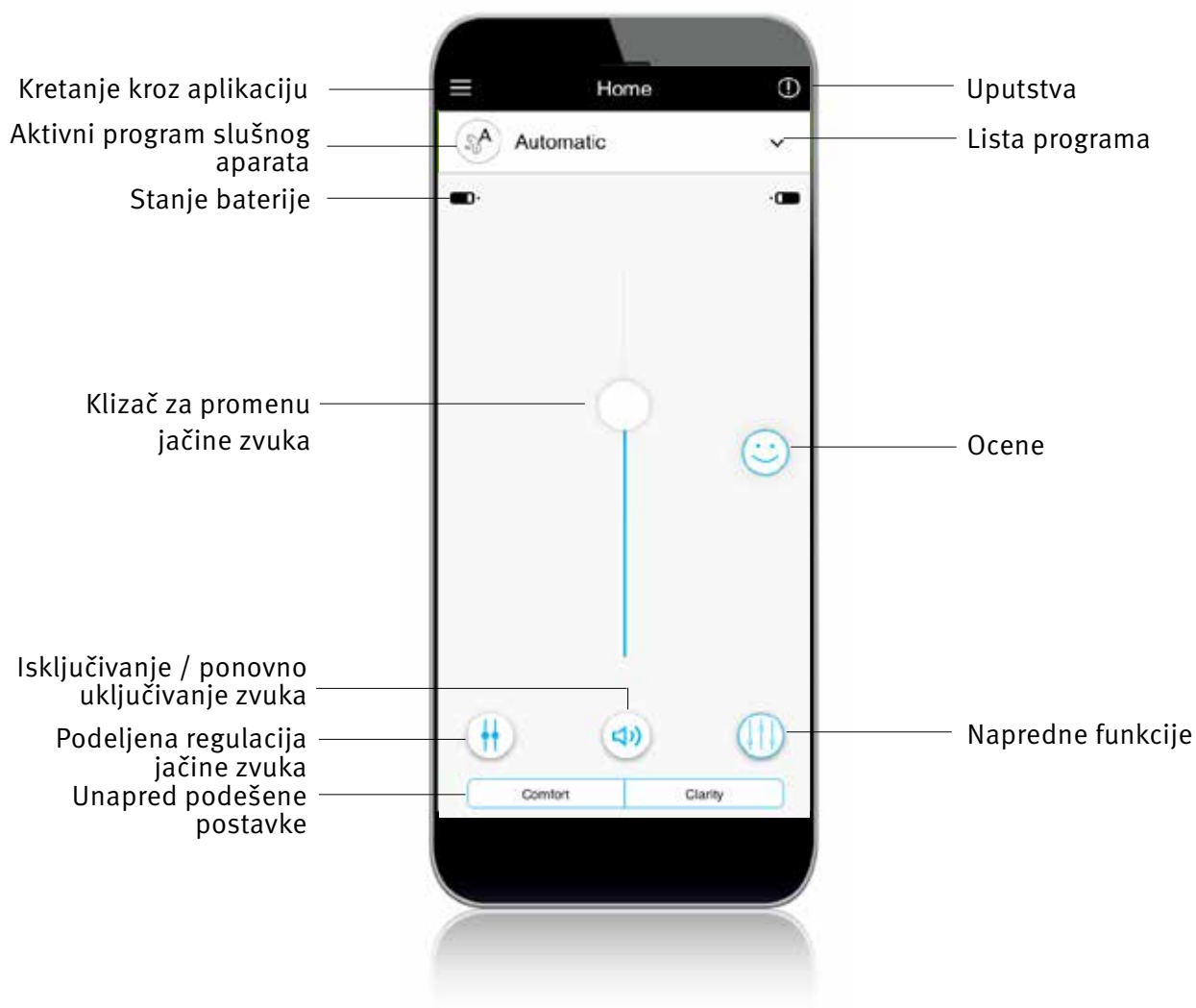
Android, Google Play i Google Play logotip su zaštitni žigovi kompanije Google Inc.

Sadržaj

Pregled aplikacije	4
Izjava o privatnosti	5
Uparivanje sa slušnim aparatima	6
Glavni ekran.....	7
Omogućavanje unapred podešenih postavki – Comfort i Clarity.....	8
Promena programa na slušnim aparatima.....	9
Podešavanja naprednih funkcija	10
Ocene	12
Meni sa podešavanjima.....	14
Opcionalni programi	17
Daljinsko podešavanje	23
Informacije o usaglašenosti.....	24
Informacije i objašnjenje simbola.....	25

Pregled aplikacije

Izjava o privatnosti

Prihvatanje izjave o privatnosti podataka u aplikaciji

Da biste koristili aplikaciju Unitron Remote Plus, morate da prihvatite izjavu o privatnosti i analizu anonimnih podataka o korišćenju aplikacije.

Privacy notice

Last updated: August 15, 2019

THIS NOTICE DESCRIBES HOW PERSONAL INFORMATION ABOUT YOU MAY BE USED AND DISCLOSED, HOW YOU CAN GET ACCESS TO THIS INFORMATION AND WHICH RIGHTS YOU HAVE. PLEASE REVIEW IT CAREFULLY.

This Privacy Notice describes the following:

- The personal data that Sonova collects from and about you through the Unitron Remote Plus application ("The Mobile Application" or "Mobile App") or Unitron TrueFit fitting software together with all related technology to access or otherwise use the Mobile Application ("Mobile Service") as described below.
- How do we use and protect your personal data?
- The choices you have about how we use your personal data.
- How you can exercise your rights (e.g. right to access, right to rectification, right to complain, etc.).

This privacy notice is in line with the EU General Data Protection Regulation (GDPR), the Swiss Federal Data Protection Act (FDP), the United States Health Insurance Portability and Accountability Act (as amended from time to time) ("HIPAA") and other applicable data protection laws that may apply to you. In the United States, any reference to HIPAA under this Privacy Policy to personal data shall apply to your personal data that fits the definition of Protected Health Information under HIPAA. Protected Health Information is any piece of information in an individual's medical record that was created, used, or disclosed during the course of diagnosis or treatment that can be used to personally identify you.

If you are unwilling to accept the terms and conditions of this Privacy Policy, we ask you to not use this Mobile Application and/or the Mobile Service. By using the Mobile Application or the Mobile Service, you agree to the terms of this Privacy Policy. When you submit information via the Mobile Service, you provide consent to the collection, use and disclosure of that information in accordance with this Privacy Policy.

We reserve the right to occasionally update this Privacy Policy. At such time, we will also revise the "last updated" date shown in this Privacy Policy. We reserve the right to apply the terms of any updated Privacy Policy to the all personal data that we maintain. We shall inform you about changes to a revised Privacy Policy by [email or by another way to be used by us](#).

Accept

Aktiviranje „Insights“ (Uvidi) funkcija

Da biste prihvatili **Insights** (Uvidi) funkcije, uključujući i funkciju daljinskog podešavanja, tapnite na dugme „**Activate**“ (Aktiviraj). Da biste preskočili ovaj korak, tapnite na dugme „**Later**“ (Kasnije).

**You're unique.
So is your hearing.**

Activate Insights to help your hearing health care professional get you the best fit.

- Rate your hearing experiences
- Track your wearing time
- Track your listening environments
- Enable remote hearing aid adjustments

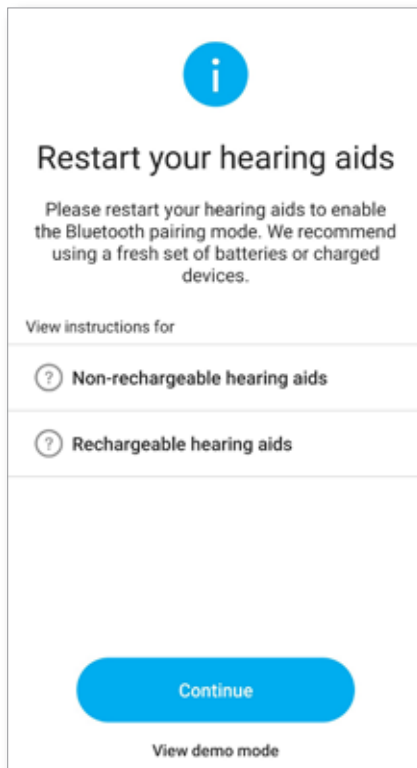
By activating insights you consent to store your personal information in the cloud for your hearing health care professional to access.

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Sonova AG
Sonova Deutschland GmbH