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Privacy Policy

The purpose of this notice is to inform you about the types of data that is collected with the app, how that data is used and with whom it is shared.

[Hide detailed policy](#)

Terms of use

This app has been developed by Sonova AG, Laubisrütistrasse 28, 8712 Stäfa, Switzerland and is intended to be used with compatible wireless hearing aids. This app sends a signal and controls the hearing aids via selected mobile devices for which this app has been developed.

Please refer to the “Disclaimer” section of this settings menu and the user guides provided with your hearing aid for information on intended use, cautions and warnings.

Privacy Policy

The purpose of this notice is to inform you about the types of data that is collected with the app, how that data is used and with whom it is shared.

What personal data is collected in the app and why?

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My Ratings Privacy Policy

The purpose of this notice is to inform you about the types of data that is collected with the app, how that data is used and with whom it is shared.

[Hide detailed policy](#)

My Ratings features of the Unitron Remote Plus app

End user terms of use

This app and the My Ratings feature have been developed by Unitron for use with compatible Unitron wireless hearing aids. The My Ratings features will gather specific data from Unitron hearing aids plus information you enter and send them to the Sonova cloud by wireless signals sent from a compatible smartphone. You may also receive periodic emails asking about your overall satisfactions with your Unitron hearing aids.

We strongly recommend that you read the user guide provided with your Unitron hearing aids, and the following Disclaimer.

We also recommend that you allow notification and automatic installation of app updates on your smartphone, so that this app functions correctly.

If you have any questions, please see the FAQ section in

[Accept](#)

[Decline](#)

Activate My Ratings

Fill in the following information to activate My Ratings.

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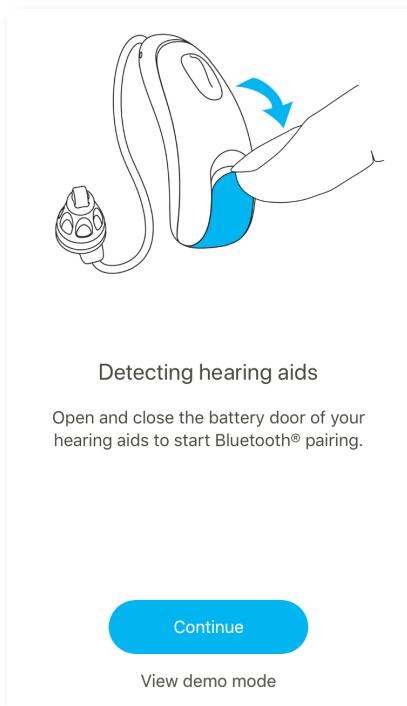
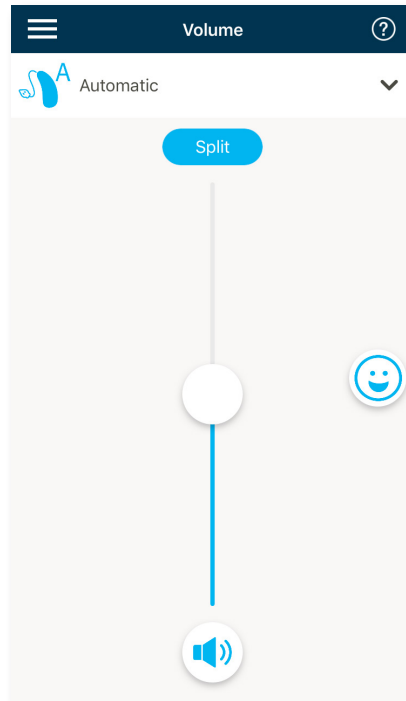
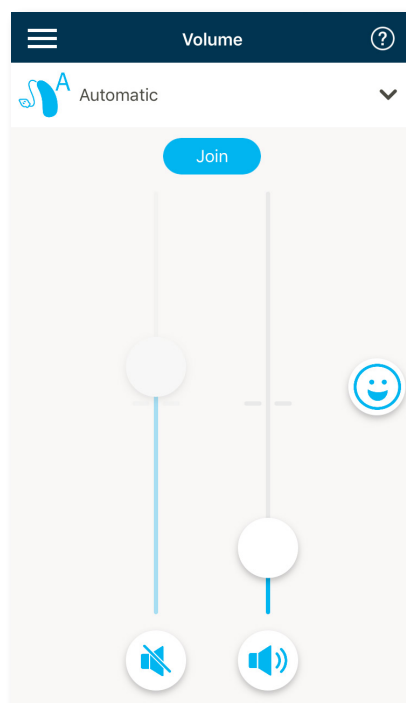
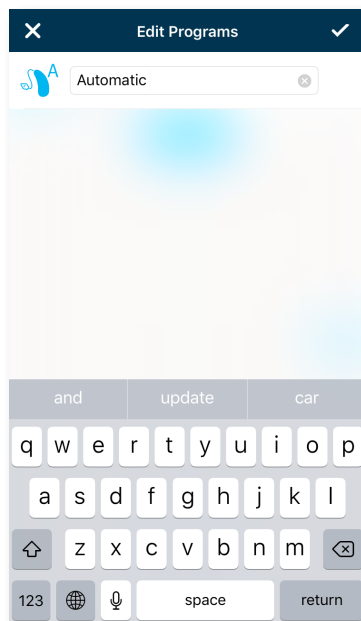
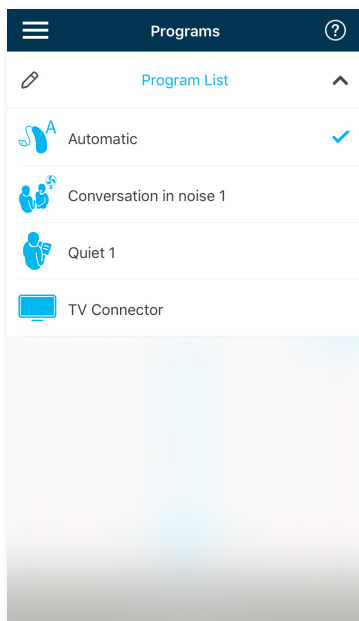
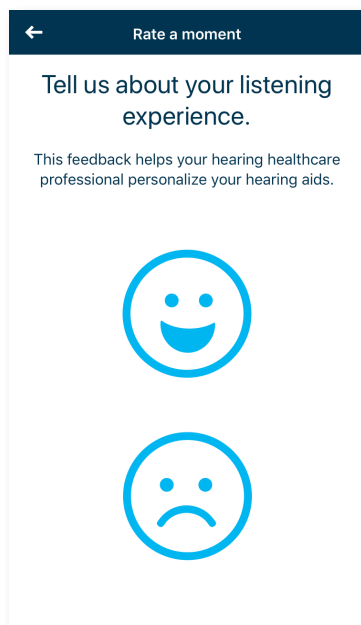
A. Nurodykite, ar esate patenkinti, ar nepatenkinti.

← Rate a moment

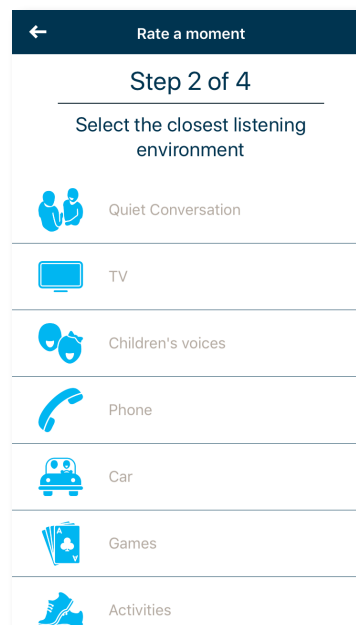
Tell us about your listening experience.

This feedback helps your hearing healthcare professional personalize your hearing aids.

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
← Rate a moment

Step 2 of 4

Select the closest listening environment

- Quiet Conversation
- TV
- Children's voices
- Phone
- Car
- Games
- Activities

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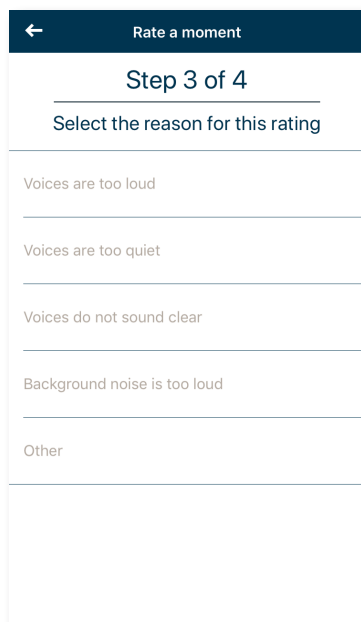
Step 3 of 4

Select the reason for this rating

- Voices are too loud
- Voices are too quiet
- Voices do not sound clear
- Background noise is too loud
- Other

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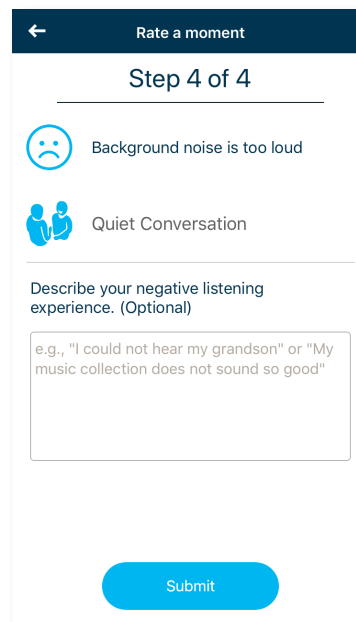← Rate a moment

Step 4 of 4

- ☹️ Background noise is too loud
- 😊 Quiet Conversation

Describe your negative listening experience. (Optional)

e.g., "I could not hear my grandson" or "My music collection does not sound so good"

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© Sonova AG, 2018. All rights reserved.