

uDirect 2 Bluetooth pairing guide

Cellular phone

Pairing and connecting uDirect 2 with your cellular phone

Charge and turn ON both your cellular phone and uDirect™ 2. Place them next to one another.

Turning on cellular phone Bluetooth®

1. Find the connectivity settings of the phone.
2. Look for **Bluetooth** under the **Menu** or one of its submenus, and turn the Bluetooth feature **ON**.

Note: If the Bluetooth settings are not under Menu, they may be under one of these: Options, Tools, Settings, Connections, Connectivity, Applications, Control Panel, Setup, or under a combination of any of these (i.e. Menu > Settings > Connectivity > Bluetooth).

Setting uDirect 2 into pairing mode


To initiate the pairing process on uDirect 2, set uDirect 2 into pairing mode by pressing and holding the (+) and (<>) buttons for about two seconds until the Bluetooth LED indicator is blue and flashing quickly. uDirect 2 is now in pairing mode for two minutes or until the pairing is complete.

Searching for uDirect 2 and confirming pairing with passcode 0000 (four zeros)

1. On your cellular phone, choose to **add a new Bluetooth device**. The phone starts searching for Bluetooth enabled devices within range.
2. Under the list of available devices, select **uDirect 2** and enter **0000** (four zeros) when asked for the passcode or passkey. Allow devices to complete the pairing process. On the cellular phone, uDirect 2 is shown as a paired device and on uDirect 2, the Bluetooth LED indicator flashes slowly, indicating that the devices are successfully paired.

Connecting uDirect 2 with your cellular phone

In addition to being paired, which is done only once, your uDirect 2 and cellular phone need to be connected. This is

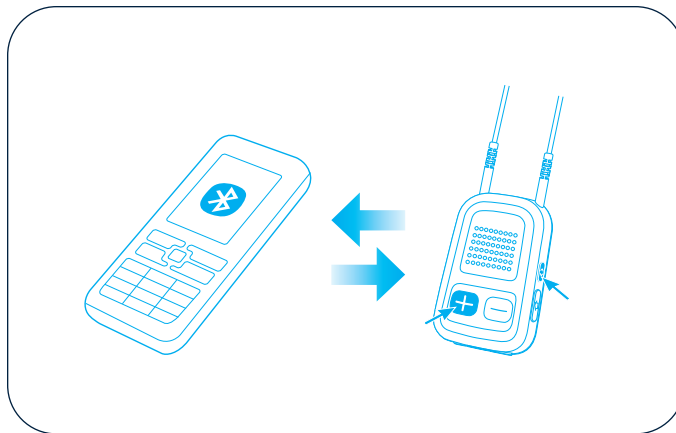

done either automatically or via the cellular phone's menu.

1. Under the Bluetooth menu, select **Paired devices** or **My devices**.
2. Highlight **uDirect 2** and select **Connect**.
3. Some cellular phones may ask which Bluetooth service you wish to enable. Select **Headset** and if available **Stereo Headphones**. On some cellular phones, the headset icon indicates that uDirect 2 is connected.
4. If prompted to allow automatic reconnection, do so. This way uDirect 2 and your cellular phone will reconnect whenever within range.

Cellular phone compatibility issues

Bluetooth functionality is implemented in different ways depending on the cellular phone manufacturer. Unfortunately, some cellular phone models with Bluetooth functionality are compatible only with certain types of devices. If a problem occurs with the connection between your cellular phone and uDirect 2, or if you are uncertain of your cellular phone's Bluetooth capabilities, consult your cellular phone user guide or ask your cellular phone dealer.