

Recommended systems

System component	Recommendation
Processor	Pentium i5, 3 GHz or faster
RAM	8 GB or more
Hard disk space	4 GB or more
Operating systems supported	• Windows Vista, 32 bit/64 bit, latest SP • Windows 7, 32 bit/64 bit, latest SP • Windows 8, 32 bit/64 bit, latest SP • Windows 8.1, 32 bit/64 bit, latest SP • Windows 10, 32 bit/64 bit, latest SP
Display resolution	1280 x 1024 pixels
Patient secondary display resolution	1366 x 768 pixels
Graphic card	16 Million (24 bit) screen colors or more
Drive	DVD-ROM
Serial COM port	One for HI-PRO if used via serial COM port

System component	Recommendation
USB ports	One for each purpose: • Bluetooth adaptor • Accessory programming • HI-PRO USB / HI-PRO 2 / iCube II
Programming interfaces	• iCube / iCube II • NOAHlink • HI-PRO / HI-PRO USB / HI-PRO 2
NOAHlink driver	NOAHlink properties and NOAHlink kernel: latest version available
Internet connection	Highly recommended
Browser	• Internet Explorer® latest version • Mozilla® Firefox® latest version • Google® Chrome™ latest version • Microsoft Edge® latest version
Sound card	Stereo
Playback system	20 Hz – 14 kHz
NOAH version	NOAH 4.5 (NOAH 3.7.1 or newer required)

unitron TrueFit™ software

Successful fittings. Simplified.

Unitron TrueFit v3.3
Installation guide

A Sonova brand

Installing Unitron TrueFit software

1. Insert the Unitron TrueFit DVD into your DVD-ROM drive. The installation program should be launched automatically by your operating system. If it is not, either:

Go to: Windows Start, choose Run... enter the file information.
(i.e. d:\setup.exe, where d: represents the letter of your DVD-ROM drive) and click on OK.

OR

Using Windows Explorer (My Computer), locate the file setup.exe on your DVD-ROM drive and double-click the file name.

2. Select the preferred installation language: English, French, German or Dutch (for the installation procedure only).



3. When the Unitron TrueFit installation window appears, follow the instructions on the screen, such as selecting the preferred installation type: NOAH or Standalone.



 Instructions are available at www.unitron.com/instr in Adobe® Acrobat® PDF format. To view them, you must have Adobe Acrobat Reader installed. Visit Adobe.com to download.

 To obtain a free paper copy of the instructions, please contact Unitron using the phone numbers or address in this guide. A copy will be sent within 7 days.

Need assistance?

Australia - 1800 212 313
Austria - +49 711 658538 0
Belgium - +31 88 600 88 10
Brazil - +55 11 3748 0960
Canada - 1 800 265 8255
China - +86 21 61205533
Colombia - +57 1 2149045
Denmark - +45 64 41 78 87
EU Representative - +41 58 928 01 01
France - 04 26 23 22 00
Germany - +49 711 658538 0
India - +91-22-26871151

Netherlands - +31 88 600 88 10
New Zealand - 0800 864 8766
Norway - +47 22 47 76 30
Russia - +7 495 788 02 04
South Africa - (011) 467 7662/52
South Korea - +82 2 859 4902
Spain - +34 902 494 960
Sweden - +46 (0) 8 546 20 960
Switzerland - 0800 928 800
U.K. - +44 (0) 1925 247810
U.S. - 1 800 888 8882

For countries not listed - +1 519 895 0100



Unitron Hearing, a division of
National Hearing Services Inc.
20 Beasley Drive, P.O. Box 9017,
Kitchener, ON N2G 4X1 Canada



sonova
HEAR THE WORLD